





# Reference Guide for ToxCup™






ToxCup™ is ideal for the collection of urine, testing for drugs and adulterants and shipping for confirmation.

## ToxCup™ Procedures

The simple instructions are as follows. Refer to package insert for detailed instructions and technical information.

<p><b>1. Remove</b> the test lid from the sealed pouch.</p> 	<p><b>2. Twist</b> the test lid onto specimen cup after collection.</p> 
<p><b>3. Lean</b> cup on side to activate testing.</p> 	<p><b>4. Read</b> results once the control bands (C) form (approximately 5 minutes). Results are stable for up to an hour after the control bands form. Interpret according to instructions below.</p> 

## Interpreting ToxCup™ Test Results

<u>Invalid Result</u>	<u>Negative Result</u>	<u>Presumptive Positive Result</u>
<p>When <b>No</b> band appears in the control (C) region, <b>the test is invalid</b> even if there is a band in the test region. Check testing procedures, samples, and/or control materials. <b>Repeat</b> the procedure with a new lid.</p> 	<p>2 Colored Bands should be observed</p> <ul style="list-style-type: none"> <li>• in the CONTROL (C) region</li> <li>• in the TEST region</li> </ul> <p>The color of the test band may be slightly darker or lighter than the control band. Any band that can be seen visually, no matter how faint, is a <b>negative</b> result.</p> 	<p>• A colored band in the CONTROL (C)</p> <p>• <b>No</b> colored band in the test region</p> <p>indicate a presumptive positive result.</p> 
<p>The above example is interpreted:                      AMP: Negative      COC: Negative                      MET: Invalid      OPI: Invalid                      THC: Negative</p>	<p>The above example is interpreted:                      AMP: Negative      COC: Negative                      MET: Negative      OPI: Negative                      THC: Negative</p>	<p>The above example is interpreted:                      AMP: Positive      COC: Positive                      MET: Negative      OPI: Negative                      THC: Negative</p>